

# G5® Applicators



Genuine G5 Applicators - We're known by our Shape and by our Threads - Symbols of Quality - Unchanged since 1957™

Individual Applicators are listed below

## Applicator 209



### Right Angle Directional-Stroking® Adaptor for Directional-Percussion™ and Directional-Stroking®

Applicator 209 is used when a full percussive effect from the entire surface of an applicator is desired. This full percussive effect is obtained because the face plate of the head assembly describes a circular motion imparted to it by an eccentric cam inside the head. By screwing one leg of the percussion adaptor into the face plate, and affixing an applicator to the other, the oscillatory effect produced by the face plate is changed into a Directional-Stroking/Directional Percussion effect.

Never use this Directional-Stroking® adaptor at speeds in excess of 35 CPS.

## Applicator 210



### Soft Sponge Rubber 2 5/8" Diameter

Applicator 210 is a small sponge applicator designed for use on children or small, sensitive body parts. It is ideal for gentle massage of tender or delicate areas.

## Applicator 212



### Soft Sponge Rubber 3 1/2" Diameter

Applicator 212 is a large sponge applicator designed for gentle massage over sensitive body parts.



### **Applicator 214**

#### **Fine sponge with Plastic Back**

Applicator 214 includes one fine sponge and the plastic backing that attaches to the head assembly.

Applicators 214FS and 214FC are fine and coarse sponge replacements designed for exfoliation. Each ordering code includes ten (10) sponges.

Applicator 214FS or 214FC - \$15



### **Applicator 215**

#### **Multiple Prong Applicator**

Applicator 215 is designed for vigorous massage and surface stimulation over large body areas.



### **Applicator 216**

#### **Four-Ball Firm Rubber**

Applicator 216 is designed for deep, heavy-pressure massage of major muscle groups such as back, thighs, shoulders and buttocks.



### **Applicator 217**

#### **Half-Ball Firm Rubber**

Applicator 217 is a large, half-ball designed for working body depressions and for localized effects in postural drainage. It is most effective when applied to the soles of the feet or hands to stimulate circulation in the limbs, and to relax muscles.



### **Applicator 222**

#### **Blunt-Tipped Firm Rubber**

Applicator 222 is designed for working body depressions and the soles of the feet and hands. It can also be used for trigger-point therapy.



### **Applicator 223**

#### **Two-Ball Firm Rubber**

Applicator 223 is used in a manner similar to applicator 216, but because of its narrower width, it fits more easily over the cervicals and is well suited to straddle the spinal column, when used for back massage. It is well-adapted for manipulation of the smaller joints, because the balls can be worked into the flesh while the oval plate connecting the balls is high enough to override intervening bony structures.



### **Applicator 225**

#### **Skin and Scalp Surface**

Applicator 225 is designed for vigorous massage and surface stimulation over large body areas and the scalp.



### **Applicator 227**

#### **Pointed-Tip Firm Rubber**

Applicator 227 is designed to reduce trigger points; for reflexology; and for wherever the thumb would be used in muscle-goading techniques.



### **Applicator 229**

#### **Large Round Firm Rubber**

Applicator 229 is 5" in diameter and designed for deep massage of the back, buttocks and thighs. Excellent for postural drainage when used with AP209 (Right Angle Directional-Stroking® Adaptor).



### **Applicator 230**

#### **Curved Flex Sponge & Frame**

Applicator 230 is designed for use on the neck, shoulders, legs, arms, back, and buttocks. Extend foam life by using AP236 or AP236D applicator covers. The Curved Flex Sponge Rubber is available by itself as AP231. The frame is also available by itself as AP232.



### **Applicator 231**

#### **Curved Flex Rubber Sponge**

Applicator 231 is the sponge part of Applicator 230.



### **Applicator 232**

#### **Curved Flex Sponge Frame**

Applicator 232 is the plastic frame part of Applicator 230. Not to be used without Applicator 231.



**Applicator 242**

**Suction Applicator**

Suction applicator for use in reducing cellulite. To be used with Applicator 209 and with the GX-99<sup>®</sup>, Cellutec<sup>®</sup>, Contour<sup>®</sup>, Fleximatic<sup>®</sup>, or TherAssist<sup>®</sup>.



**Applicator 250**

**Vibration Applicator**

For use only with the Flimm-Fighter<sup>®</sup> or Vibracare<sup>®</sup>.



**Applicator 251**

**Adult Soft Cupping Applicator**

For use only with the Flimm-Fighter<sup>®</sup> or Vibracare<sup>®</sup>.



**Applicator 252**

**Pediatric Soft Cupping Applicator**

For use only with the Flimm-Fighter<sup>®</sup> or Vibracare<sup>®</sup>.



**Applicator 253**

**Soft Foam Applicator**

For use only with the Flimm-Fighter® or Vibracare® .



**Applicator 331**

**Adaptor Ring**

Applicator 331 is designed to protect applicator threads against wear and tear. This applicator is included with Applicator 209 (Right Angle Directional-Stroking® Adaptor).



**Applicator 509**

**Micro Right Angle Directional-Percussion® Adaptor**

Applicator 509 is used when a full percussive effect from the entire surface of an applicator is desired. By screwing on leg of the percussion adaptor into the faceplate, and affixing an applicator to the other, the oscillatory effect produced by the faceplate is changed into a full percussion effect.

*For use with the G5® Microhead™ only.*



**Applicator 510**

**Micro Soft Sponge 1 3/4" Diameter**

This micro polyurethane sponge applicator is designed for use on children, elderly, or sensitive areas of the body. For postural drainage this applicator may be attached to Applicator 509 for a full percussive effect.

*For use with the G5® Microhead™ only.*



**Applicator 514**

**Fine Micro Sponge with Plastic Back**

This is a plastic back with Velcro attached upon which to affix either a soft sponge or coarse sponge. These sponges are easily replaced. A reusable micro cover helps to extend the usefulness of the sponge.

Applicators 514FS and 514FC each contain ten (10) fine or coarse sponges - \$13 each.

*For use with the G5® Microhead™ only.*

**Applicator 522**



**Micro Rounded Firm Rubber**

This rounded one-ball applicator provides a firmer touch than Applicator 510 for postural drainage and is ideal for relaxing muscles in difficult to reach areas.

*For use with the G5® Microhead™ only.*

**Applicator 525**



**Micro Scalp and Skin Surface**

This multi-fingered applicator stimulates the circulation of the skin and provides a gentle relaxation massage. This applicator is not intended for use in postural drainage.

*For use with the G5® Microhead™ only.*

**Applicator 529**



**Micro Flat Firm Rubber**

Like the macro applicator 229, this micro applicator is designed for postural drainage. This applicator may be affixed to Applicator 509 percussion adaptor for full percussive effect.

*For use with the G5® Microhead™ only.*